

Sante DICOM Viewer Lite

Series comparison guide


Copyright © 2025 [Santesoft LTD](#), all rights reserved

Table of Contents


Compare images of different series.....	2
Compare images of the same series	6

Compare images of different series


Open the series and then from the **Tab control** select the **Series View**.

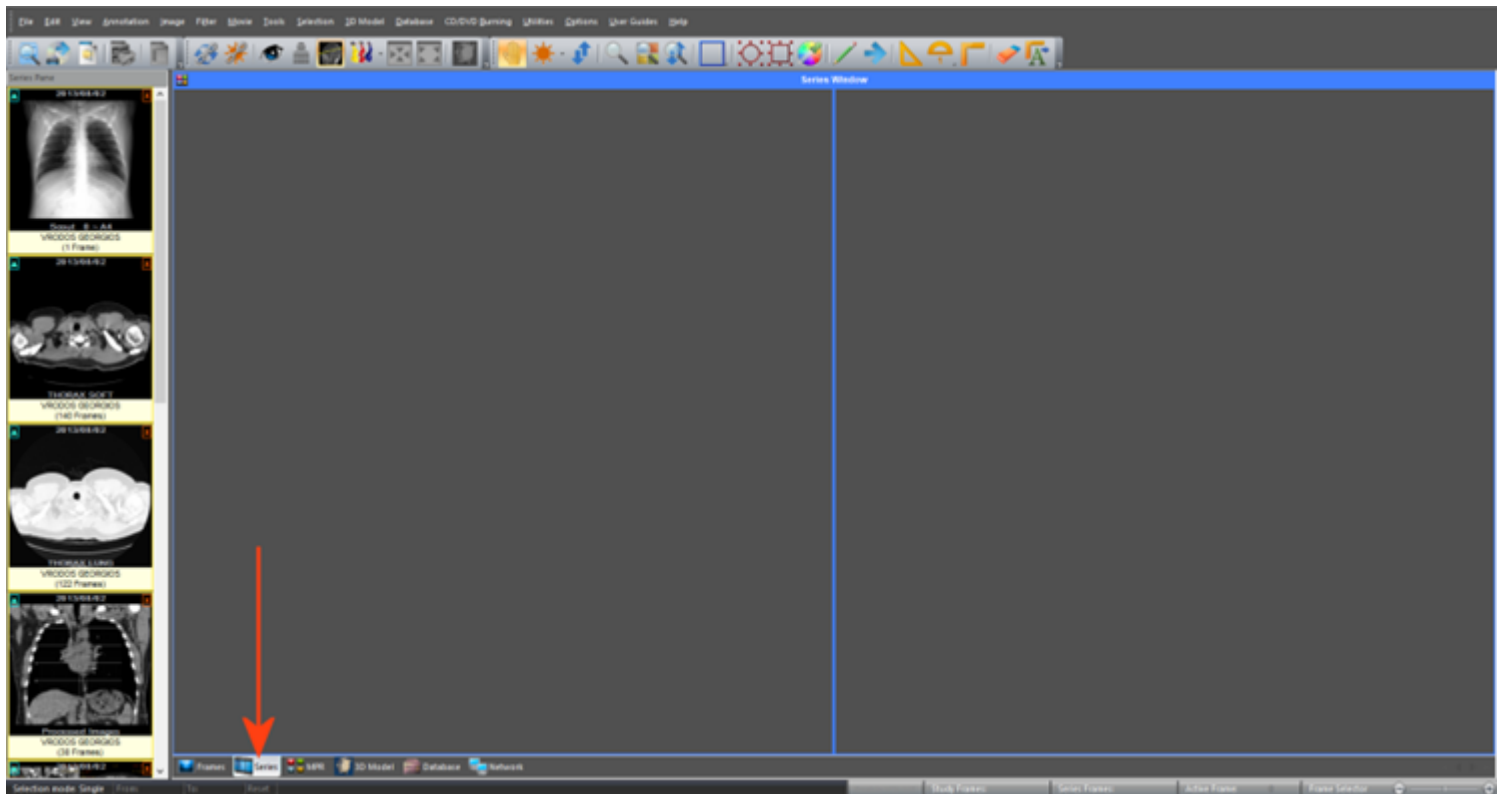


If there are already series loaded in this window, press the **right mouse button** and select the **"Empty All Tiles"** command.

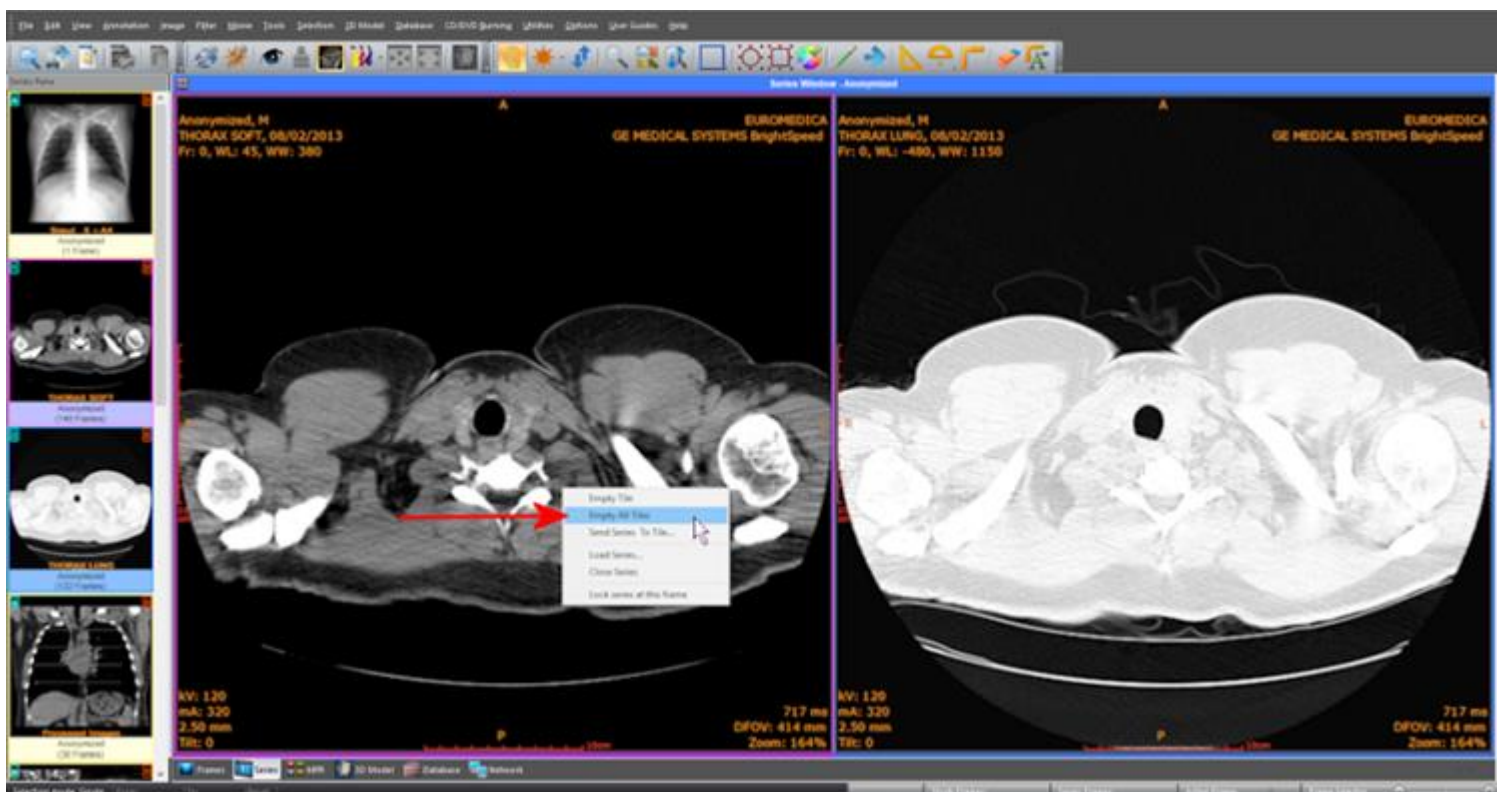



Select the menu command "**Options** ➤ **Tiles Layout**" or press the "**Tiles Layout**" button in the upper left corner of the window.

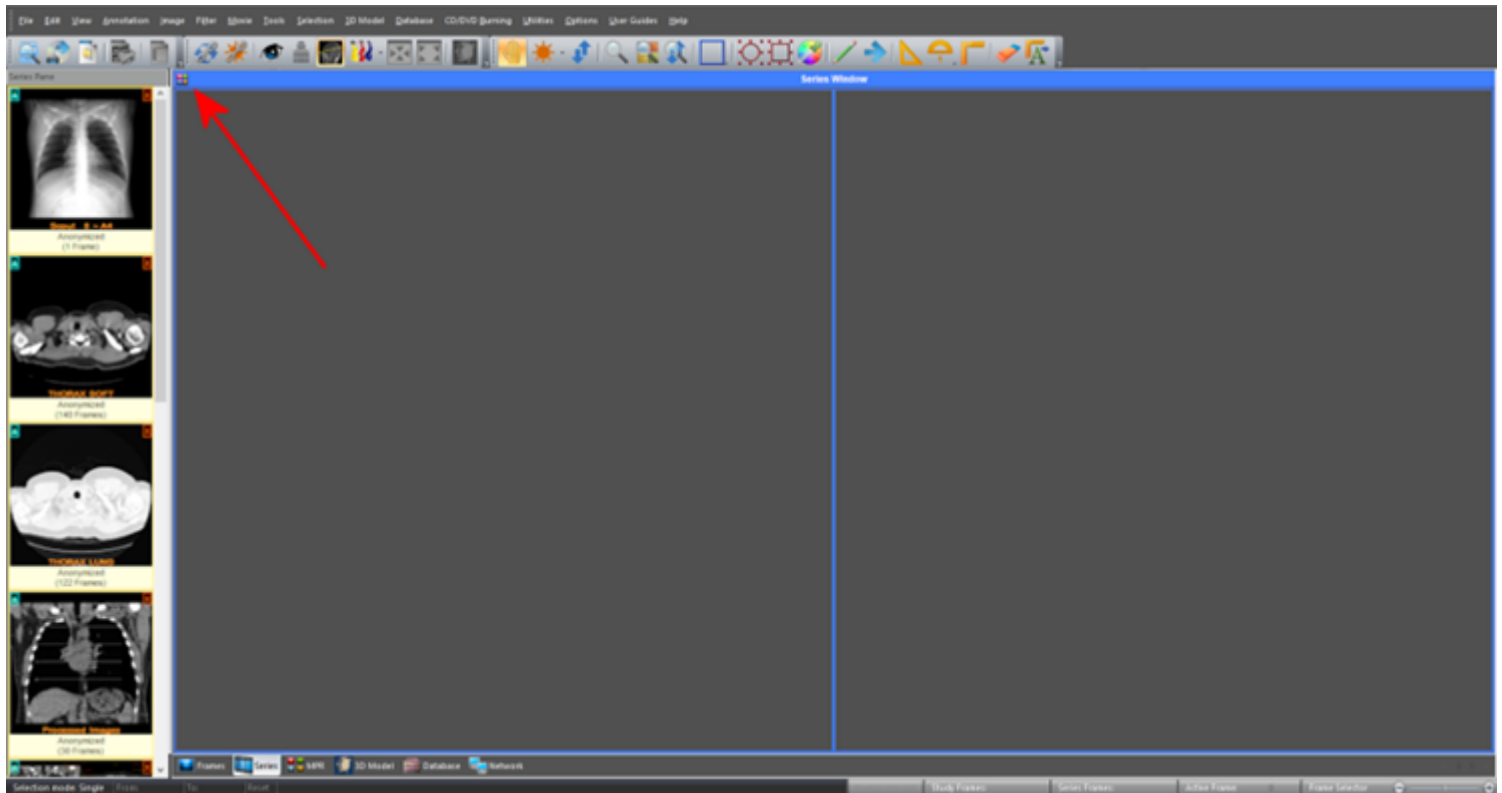


Select the 2 vertical window layout.

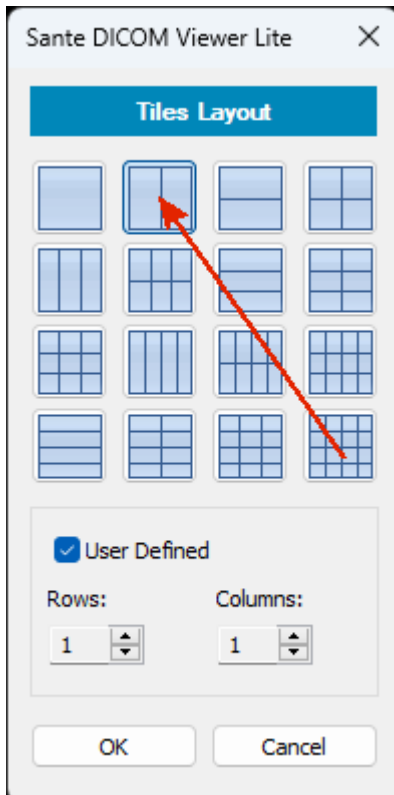

"Drag 'n' drop" the desired series into two views.

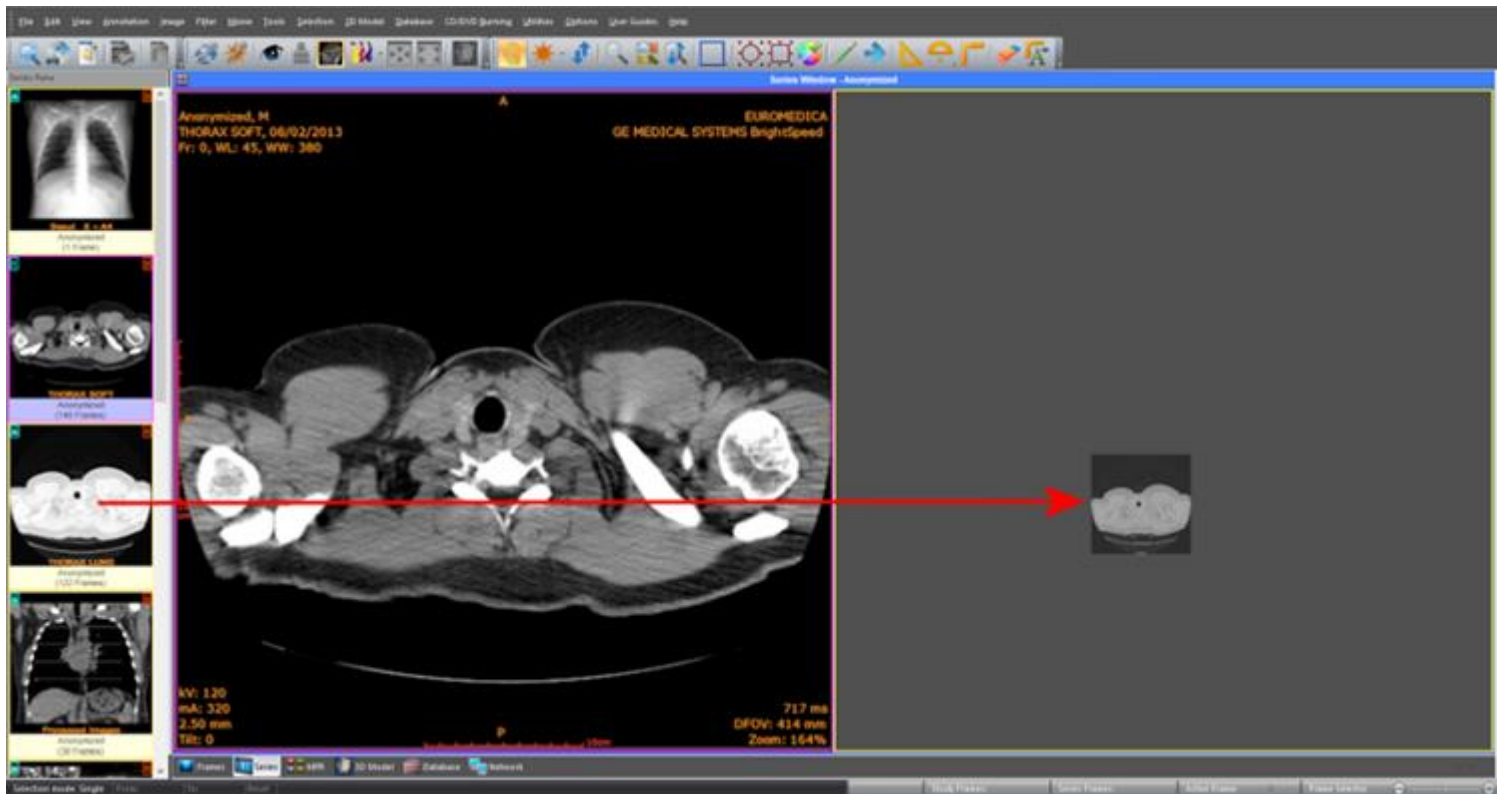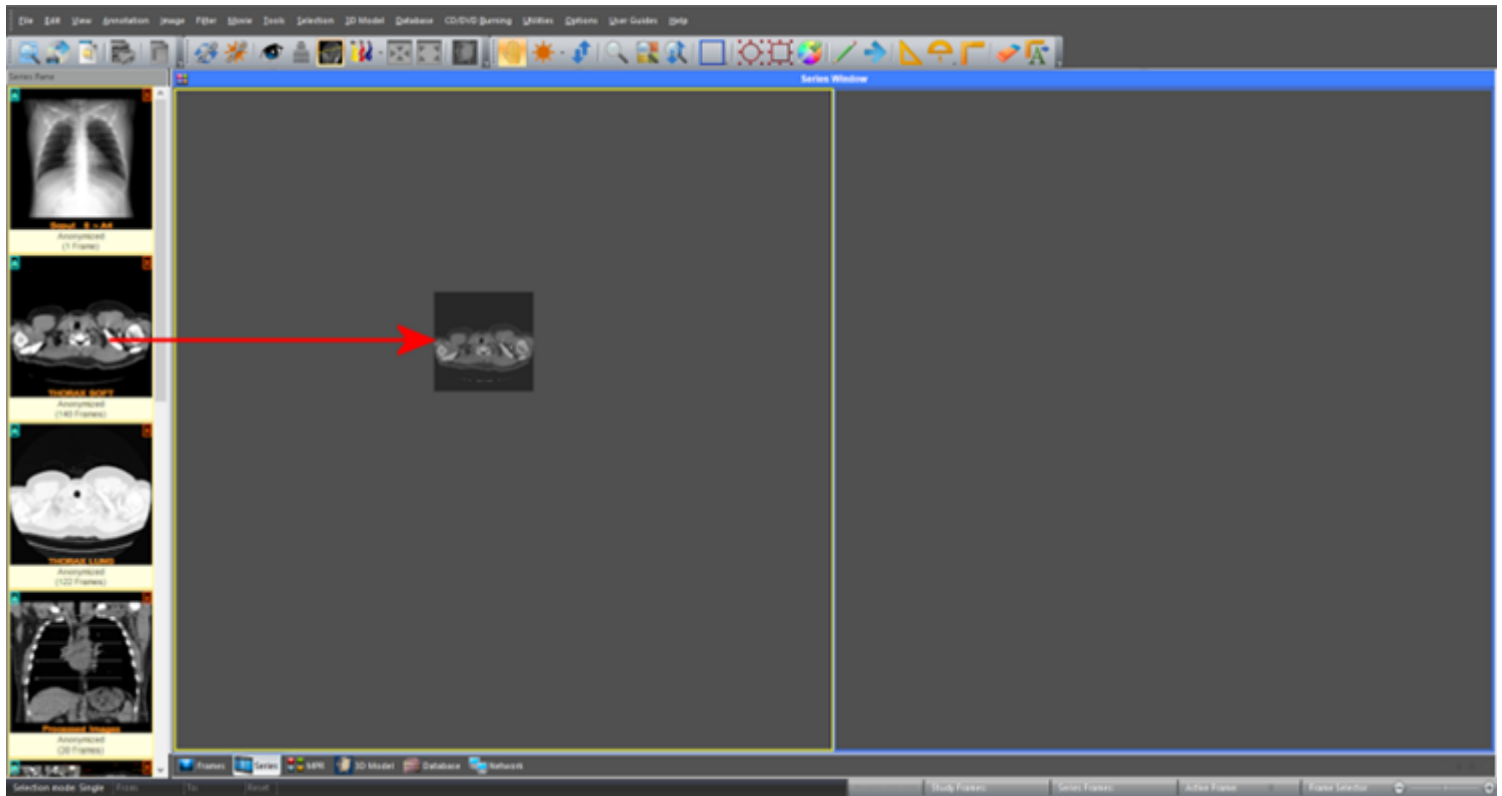


You can synchronize the frames of the two series by selecting the menu command **"View → Synchronized Series Frames"** or by selecting the corresponding toolbar button.

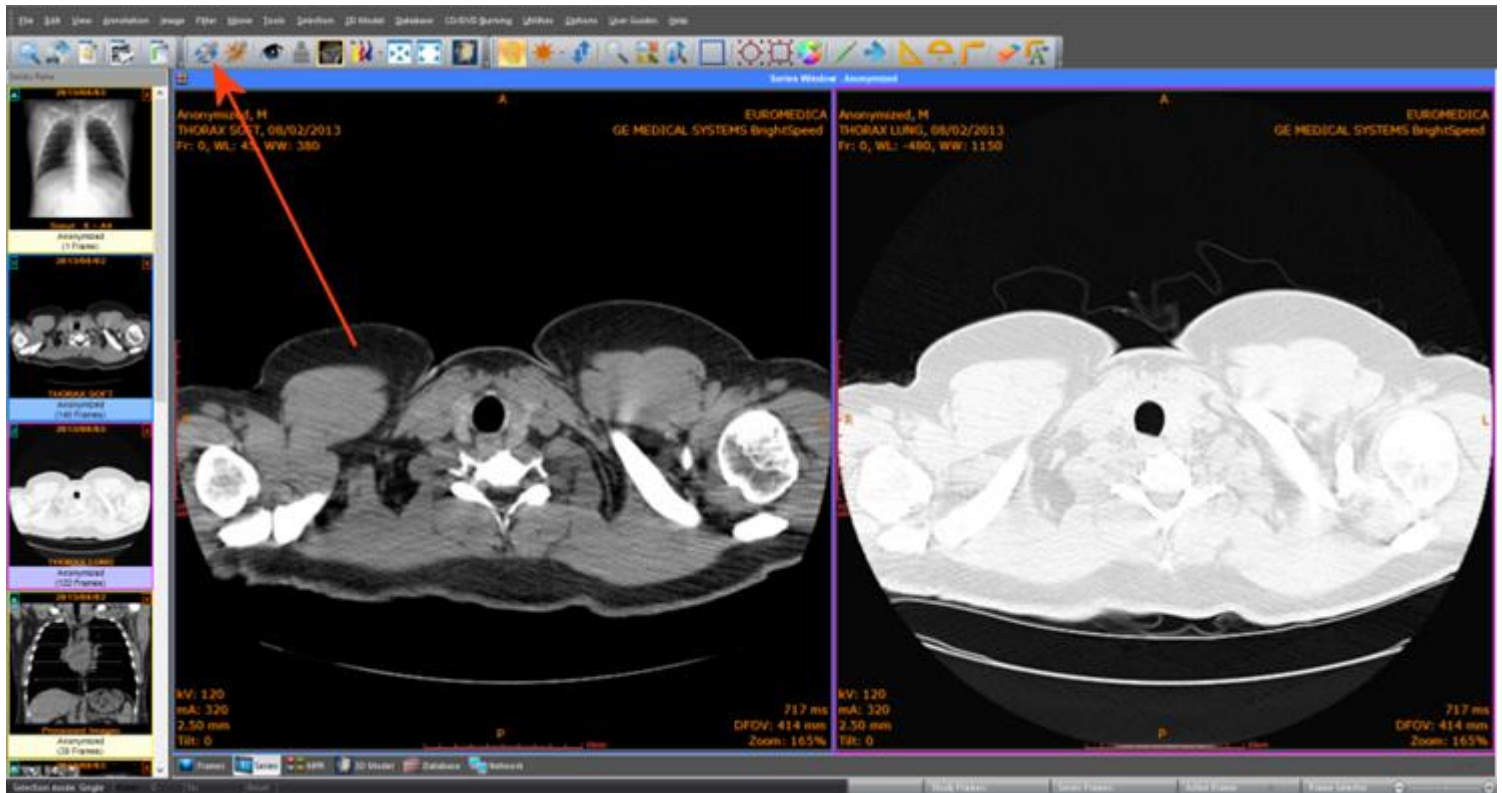


Compare images of the same series


Open the series and then from the **Tab control** select the **Series View**.

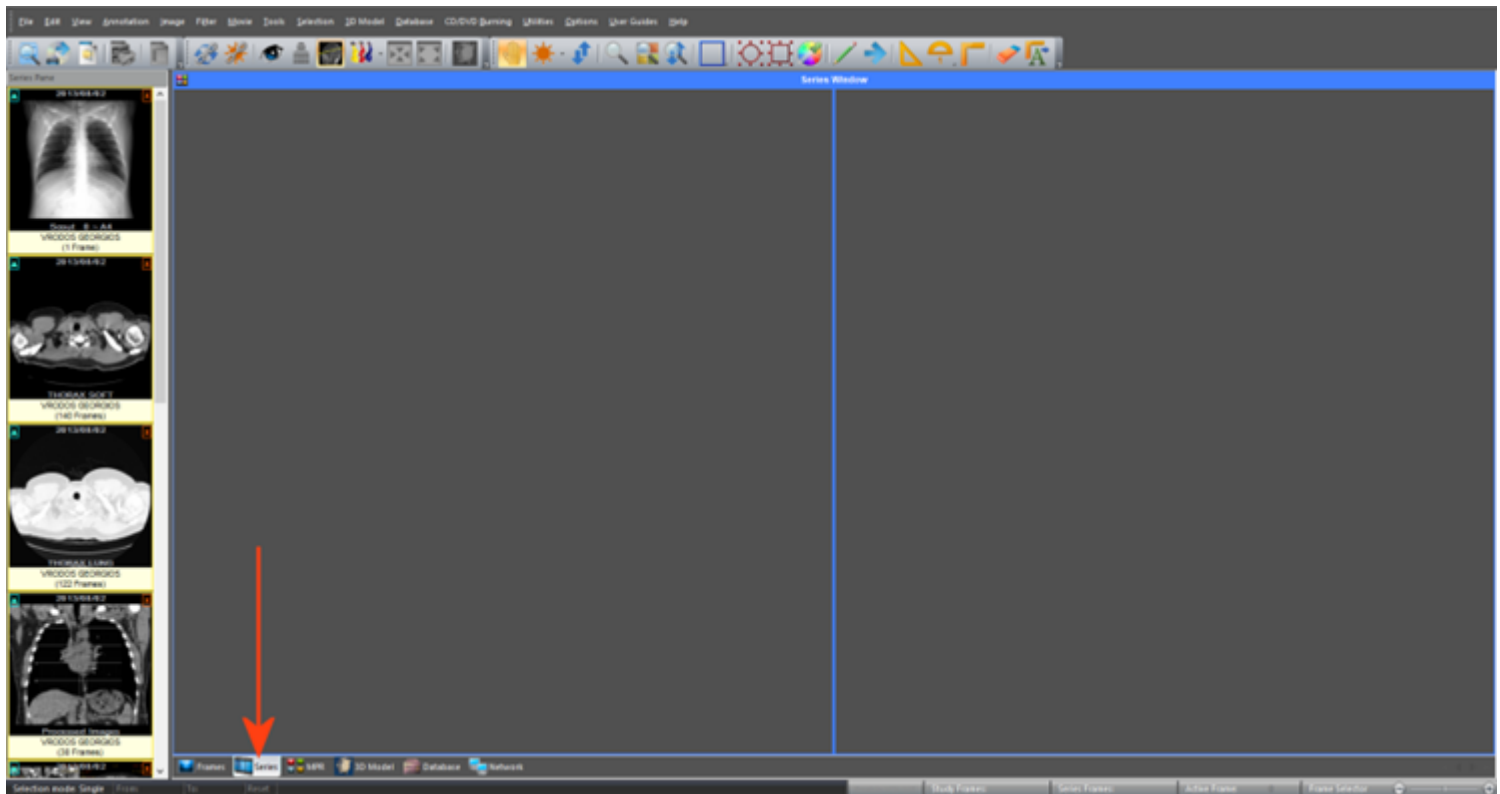


If there are already series loaded in this window, press the **right mouse button** and select the **"Empty All Tiles"** command.

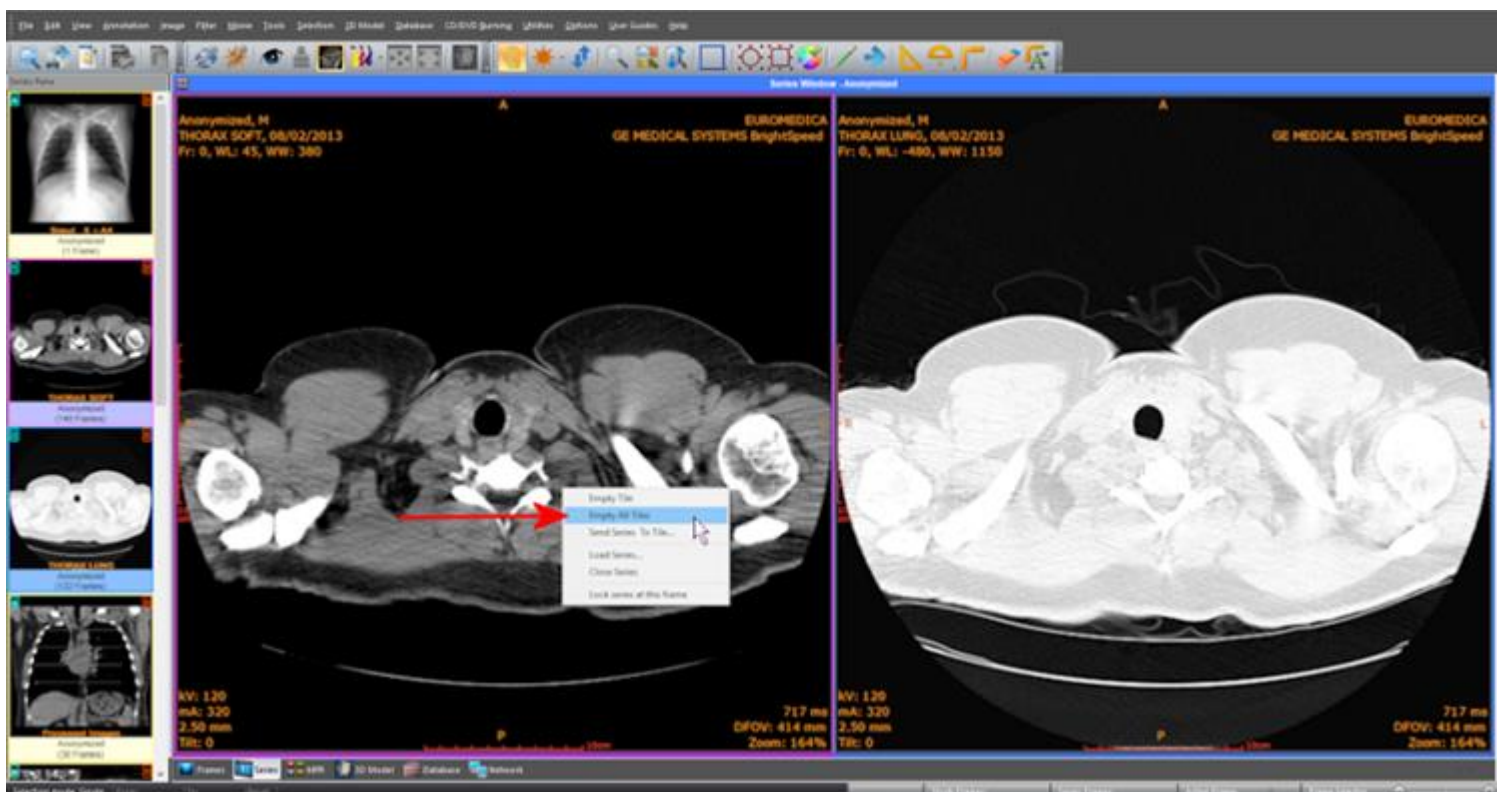



Select the menu command "**Options** ➤ **Tiles Layout**" or press the "**Tiles Layout**" button in the upper left corner of the window.

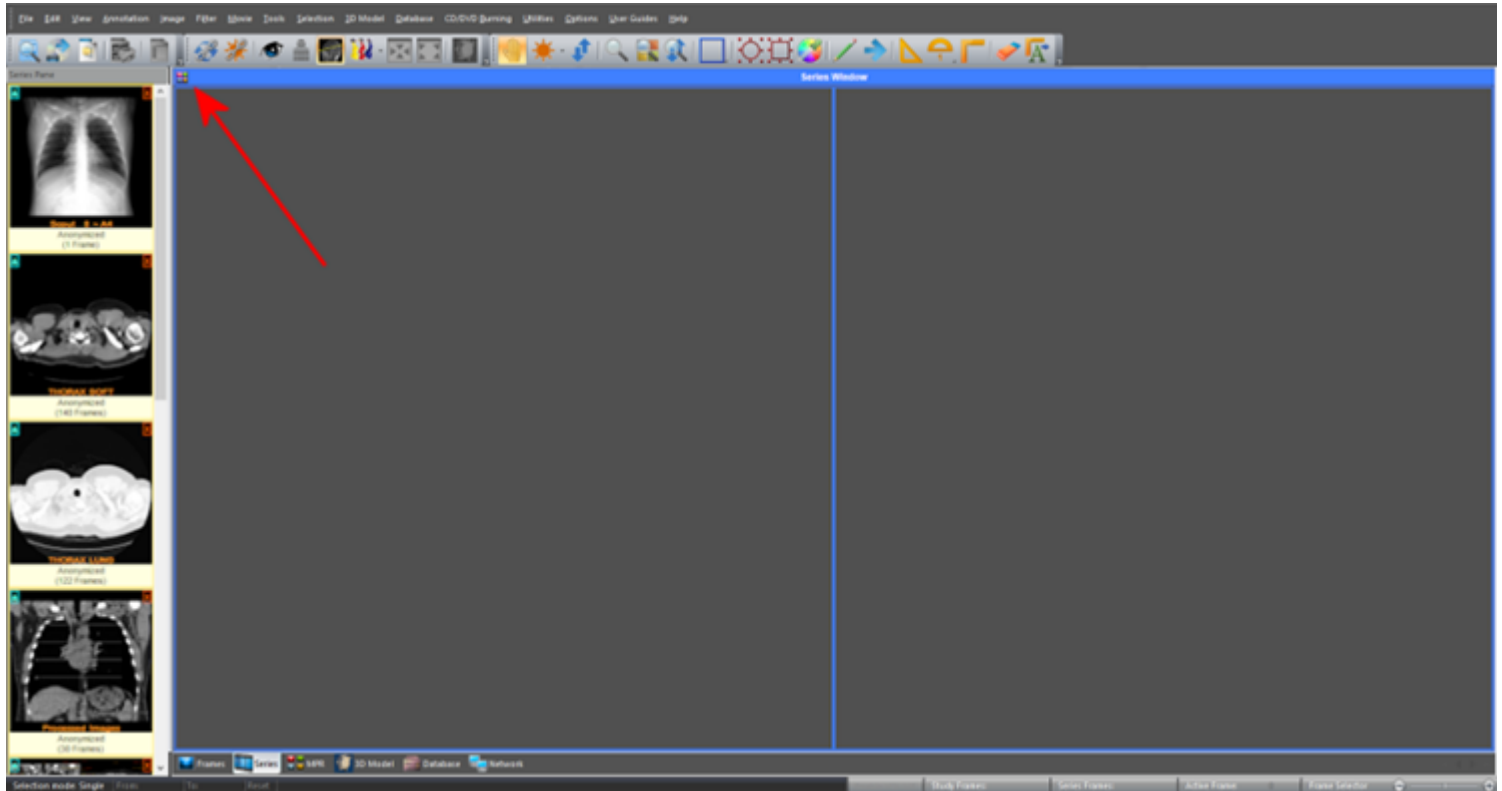


Select the 2 vertical window layout.

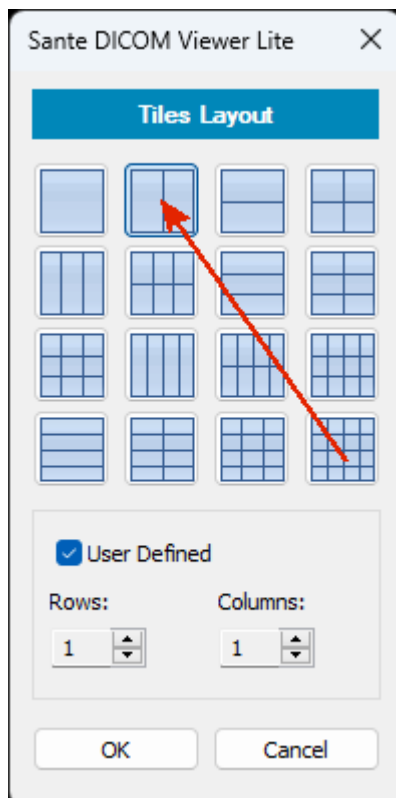


"Drag 'n' drop" the same series into two views.

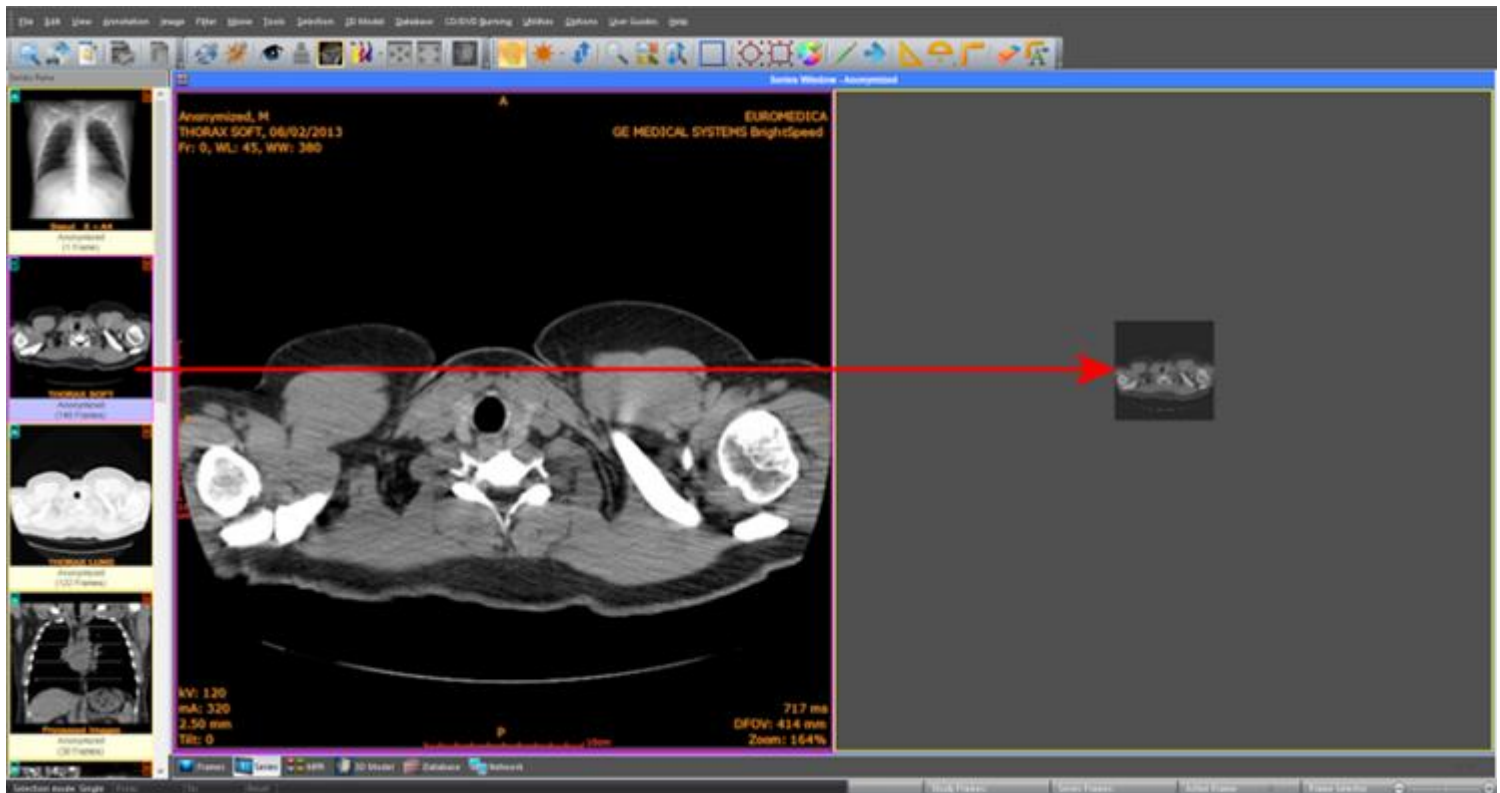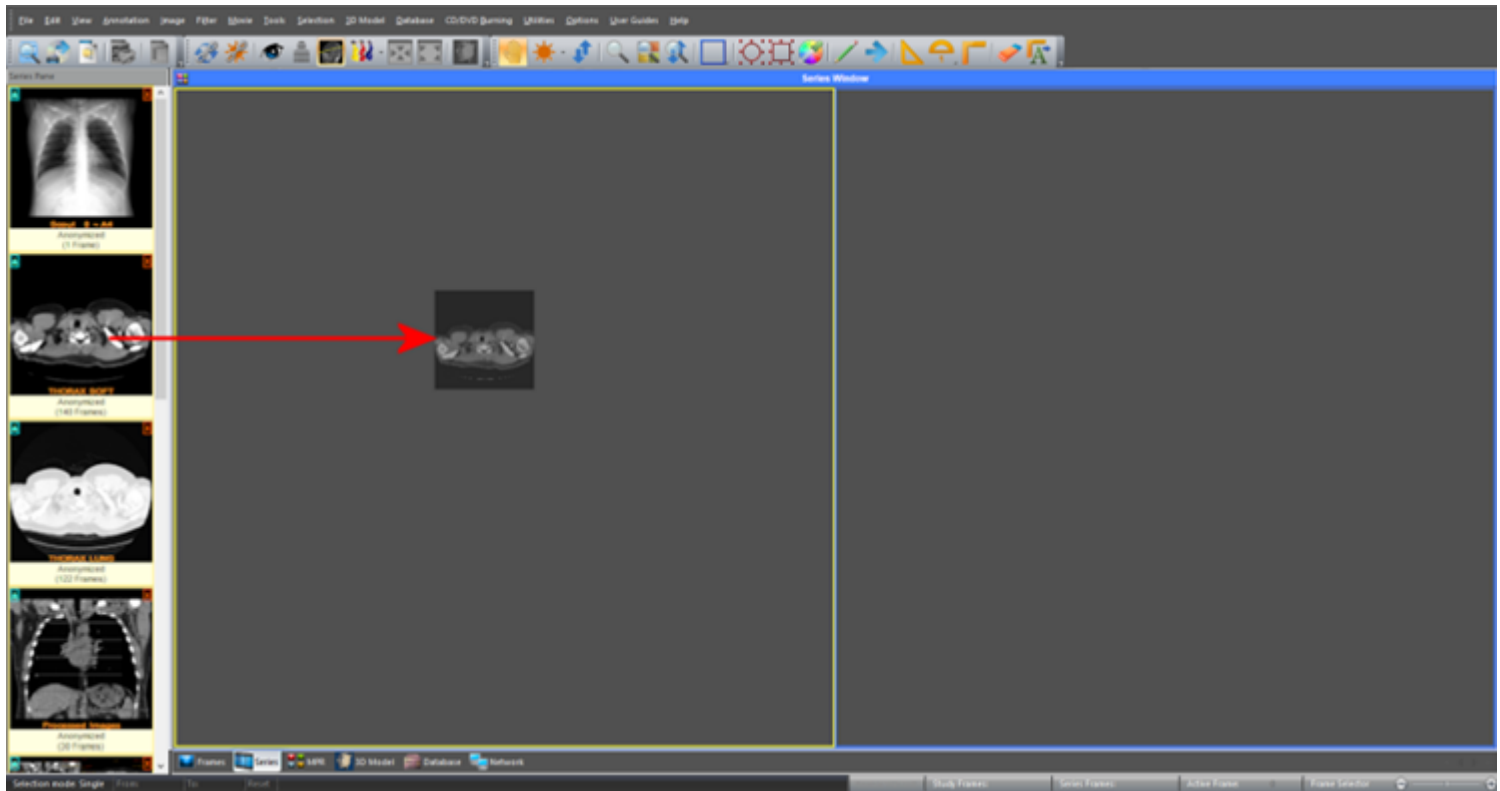

Scroll the series in the desired frame, press the **right** mouse button and select the menu command "**Lock series at this frame**".

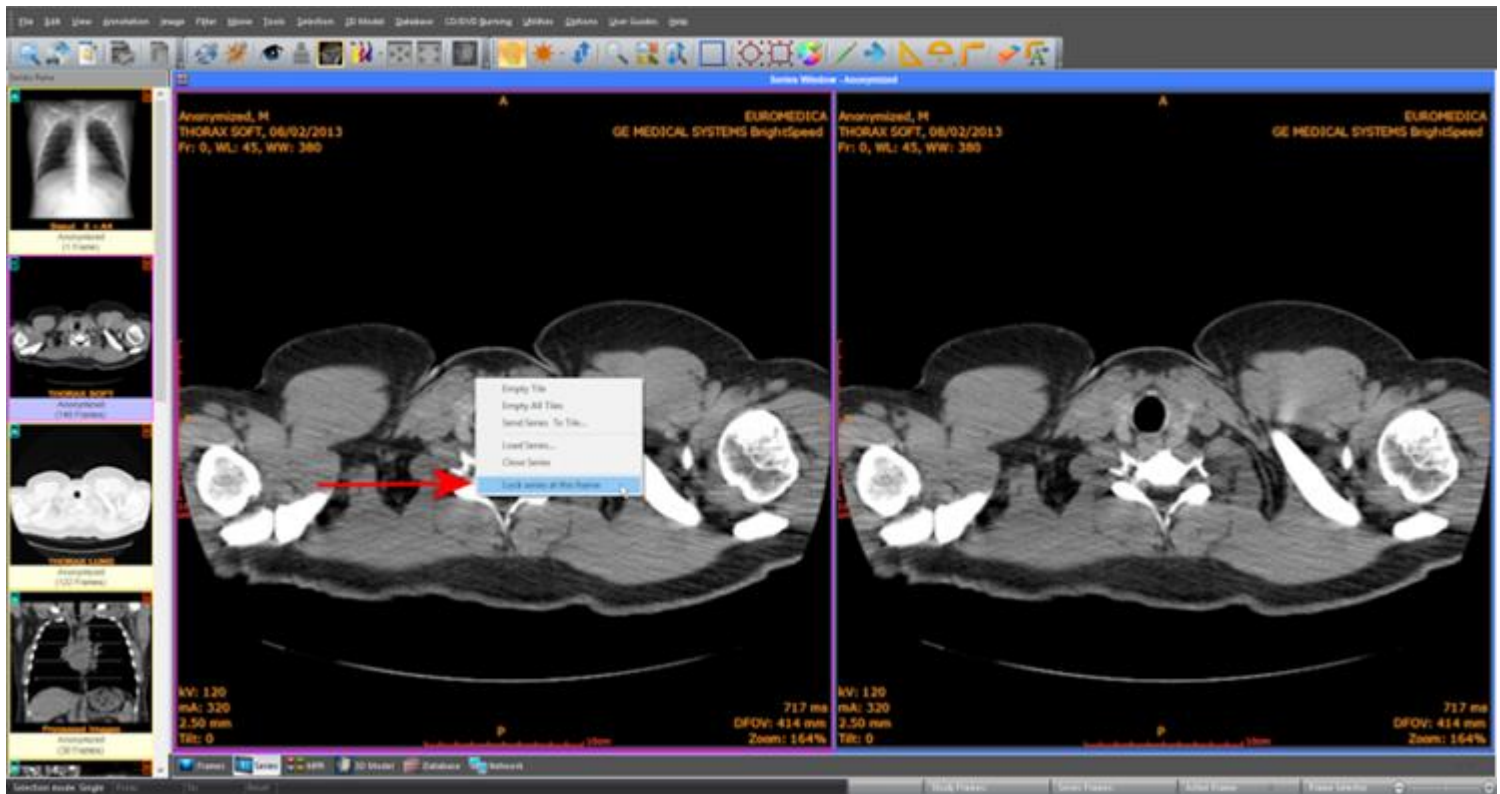

Now the one series view is locked in the desired frame, and the other series view can scroll freely to allow you the comparison of different frames. The locked series displays a lock symbol in the upper right corner of the window.

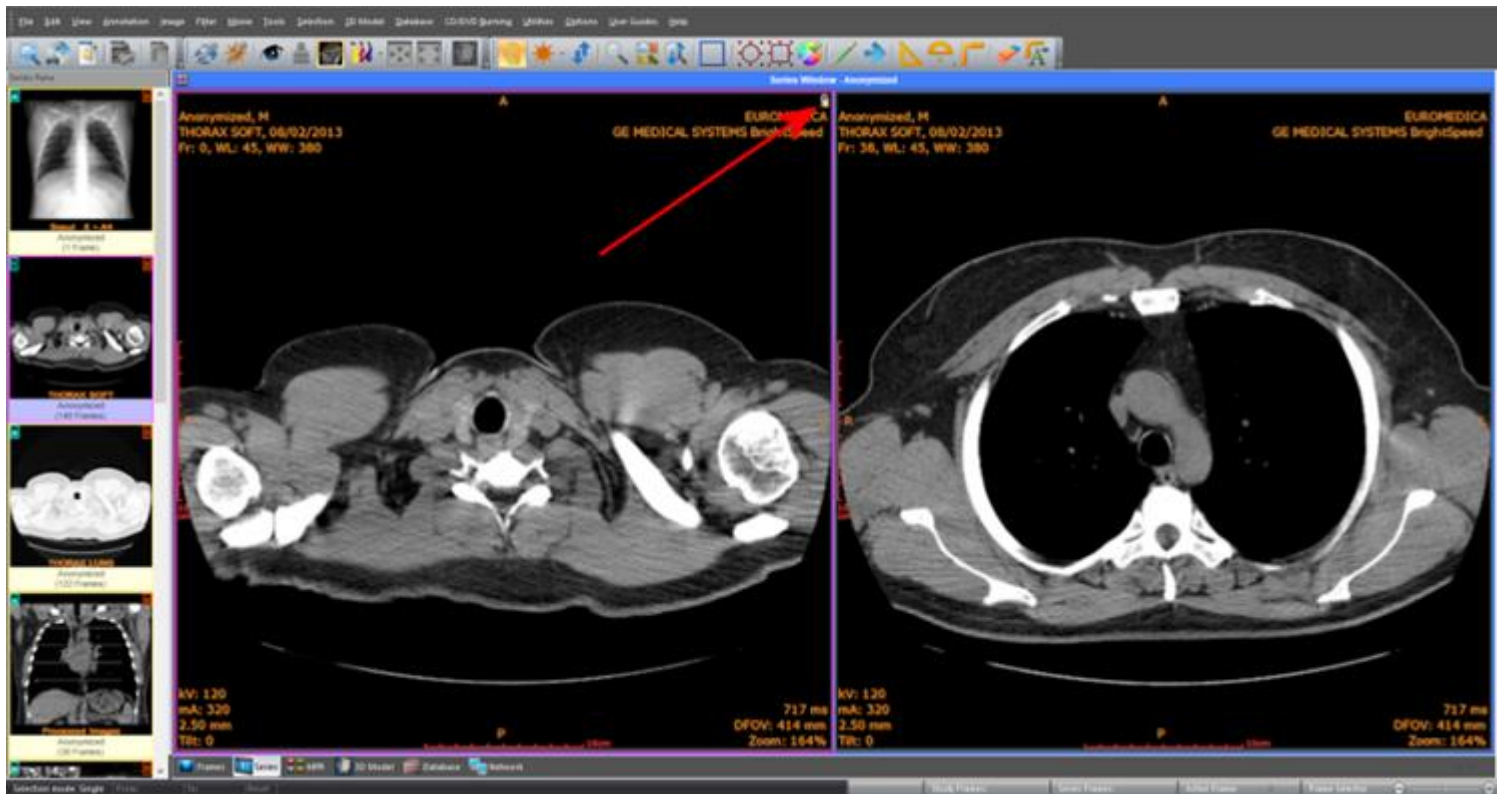

To unlock the series, press the **right** mouse button and select again the menu command "**Lock series at this frame**".

